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
INDERMIL® flexifuze™ has been used to close your wound, the information contained within this leaflet will help you understand this closure method.

Product

Indermil® flexifuze™ is a tissue adhesive and can be used as an alternative to other closure methods such as sutures and staples.

Indermil® flexifuze™ is applied to the skin as a liquid which immediately begins to harden on contact with the surface of the skin. When fully dry, the adhesive will appear wrinkled, this is perfectly normal.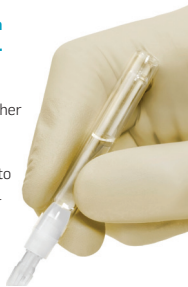

Wound treatment

The wound should be kept as dry as possible until the adhesive barrier sloughs off in 5 – 8 days allowing for only occasional contact with water. Patients may immediately take a short shower and bathe the wound site gently. Avoid scrubbing the wound site directly or excessive soaking in water (including swimming) until the adhesive barrier sloughs off and the wound site heals. Do not pull or pick at the wound or bandage. Do not apply any medications or cream to the wound. Keep the wound dry with a non-stick, non-medicinal and water resistant bandage, per your doctor's instructions.

Report any discomfort or other concerns regarding your wound to your doctor.